

HYGIENE COVER inmuRELAX

Can be disinfected with 70-85% ethanol or washed up to 95°C



Recommendation:
*Desinfect the inmu and the
hands of the user before use!*

Desinfection and wiping



- The cover and core can withstand being wiped with alcohol (70-85%), including a disinfectant wipe.
- The cover and core are water repellent and can withstand wiping with a damp cloth.
- Caution! The inmu is NOT waterproof.

Washing of the hygiene cover



- The hygiene cover can be machine washed at up to 95°C. Always follow the washing instructions.
- To keep the cover neat, it can be wiped with a dry cloth after washing.

Charging and changing the volume

- Remove the cover so that the charging plug is visible and connect the charging cable.




Here you see the C-fold and the A-fold dismantled while the inmu is being charged.


Put on the hygiene cover

Note that the shape of the inmu is oval and not circular. To get the right fit and to make sure that the cover will not break in the seam, please follow the mounting steps below:

Fold A: Half fold

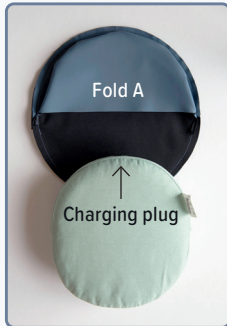

Fold B: Without logo

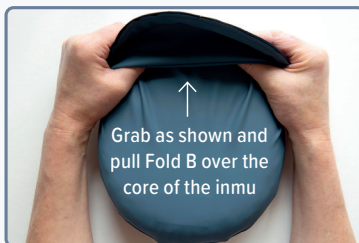

Fold C: With logo



Watch the video guide for the cover assembly: <https://youtu.be/ROpDfJAXdKI>

The benefits of the inmuRELAX hygiene cover

The surface of the cover can be disinfected with 70-85% ethanol, which makes it an easy way to protect the inmu-user from bacteria and virus. The inmuRELAX hygiene cover seals the inmu-core and protect it from liquid and impurities.

As the possibilities of tactile sensory stimulation of the inmu are reduced by the use of the hygiene cover, we recommend using this cover only for special hygiene requirements and needs.

The hygiene cover can be used when it is not appropriate to use the original cover of the inmu due to the presence of liquids and other impurities (blood, urine, etc.).

For example:

- At the dentist.
- In the hospital - where there is liquid and impurities nearby, because of viruses and bacteria or because the environment must be sterile.
- In institutions - because of the presence of liquid and impurities, or because of viruses and bacteria, or because several users use it together, for example in activities.

NOTE: Avoid dipping or spraying it with water!

The inmu with the hygiene cover on is water repellant but not waterproof.

NOTE: The hygiene cover cannot be used in combination with the other covers of the inmu, but is applied as a separate cover on the core of the inmu.



Recommendation:

Desinfect the inmu and the hands of the user before use!